

Introducer Needle [INSTRUCTION FOR USE]

1. INTENDED PURPOSE

The product is used for body puncture to collect human samples, inject liquid and gas, etc., or be a path for other devices entering the body by professional physician.

2. CONTRAINDICATION

No absolute contraindications when used correctly by medical personnel.

3. HANDING AND STORAGE

Stored in the relative humidity of not more than 80%, non-corrosive gases, a common, dry, well-ventilated and clean environment.

4. STRUCTURE AND SPECIFICATIONS

The device is consist of needle hub,needle tube and protective sleeve.

Specifications:

Specifications	Length
BR-IN-1770	70mm
BR-IN-1865	65mm
BR-IN-1870	70mm
BR-IN-2038	38mm
BR-IVC-2032	32mm
BR-IN-2138	38mm
BR-IN-2140	40mm
BR-IN-2150	50mm
BR-IN-2170	70mm
BR-IN-Y-1865	65mm
BR-IN-Y-1870	70mm
BR-IN-Y-2038	38mm
BR-IN-Y-2138	38mm
BR-IN-Y-2150	50mm
BR-IN-Y-2170	70mm

5. PRODUCT PERFORMANCE

- 5.1. Thin wall, large bore with polished inner surface
- 5.2. High quality 304 Stainless Steel
- 5.3. Sharp needle tip for lower penetration force
- 5.4. Bevel up orientation

6. INSTRUCTION FOR USE

1. Remove the packaging, do not use if the package damaged.
2. Determine blood vessel to be punctured, and then sterilize skin surface according to normal method.
3. Take off the protective needle cover.Tighten patient skin with left hand, hold seldinger needle with right hand, insert needle into the vessel rapidly from the top with an angle of 30-45 degree between skin and needle.
4. Fix the needle conventionally, and then begin to diagnose and treat.

7. PRODUCT LABEL AND PACKAGING (See the label or packaging)

- 1.Product name, model and specification.
- 2.Production date, batch lot.

- 3. Product license, registration number, product technology requirement.
- 4. Sterilization expiry date.

8. CAUTION

- 1. Procedure must be performed by trained medical personnel.
- 2. Sterilization is valid for three years, please confirm date of sterilization before use, do not use if expiry.
- 3. Do not use if luer taper leakage.
- 4. The device is designed and intended for SINGLE USE ONLY.
- 5. Do not resterilize and/or reuse, otherwise patients could get the risk of infection.
- 6. Sterilized by ethylene oxide. Do not use if the package damaged.



Caution



Use by date



Keep dry



Keep away from sunlight

REF

Catalogue number



Date of manufacture



Manufacturer



Do not use



Sterilized Using EO



Do not use if package is damaged



Batch code



BrosMed Medical B.V.

ADDRESS: Mgr. Buckxstraat 8, 6134 AP Sittard, the Netherlands

TEL: +31 850140901

Mail: cs@brosmed.com



Brun Medical Co., Ltd.

ADDRESS: Building B8, Entrepreneurship and Innovation Park, 88 Changtang Road,
Yunlong Demonstration Zone, Zhuzhou 412000 HUNAN, CHINA

TEL: +0731-22029666

WEB: www.brunmd.com

Date of issue: 2022/05/05