

Radial Artery Tourniquet [INSTRUCTION FOR USE]

1. INTENDED PURPOSE

The device is intended for use with hemostasis after the radial arterial catheterization by professional physician

2. CONTRAINDICATION

- Allergic to product ingredients
- localized skin inflammation or ulceration
- Platelet and coagulation disorders
- Retroperitoneal bleeding

3. HANDLING AND STORAGE

Stored in the relative humidity of not more than 80%, non-corrosive gases, a common, dry, well-ventilated and clean environment.

4. STRUCTURE AND SPECIFICATIONS

Snap-on(BR-RAT1) and Spiral(BR-RAT2) :The device is consist of the fastened panel, fixation band and elastomer.
(BR-RAT3):The device is consist of compression balloon and supporting board,inflatable pipe and inflatable syringe.

Specifications:

| |
|---------|
| BR-RAT1 |
| BR-RAT2 |
| BR-RAT3 |

5. PRODUCT PERFORMANCE

Designed with spiral slide to staunch bleeding,can adjust compression pressure.Suspending bracket design is able to avoid obstruction of venous reflux effectively.Pressure in air balloons is easily adjusted by specially designed syringe(RAT3).

6. INSTRUCTION FOR USE

1. Remove the packaging, do not use if the elastomer fall off or the package damaged.
2. Fixation band to assemble into the fastened panel, adjust the fixation band.(BR-RAT1)
3. Put the elastomer of the device on the puncture point, fix the fixation band, then rotary the knob to adjust the pressure.(BR-RAT2)
- 4.Inflatable syringe is connected with inflatable pipe and put the compression balloon on the puncture point,then inject gas to adjust the pressure(BR-RAT3)
5. Single use only, discards after use.

7. PRODUCT LABEL AND PACKAGING (See the label or packaging)

- 1.Product name, model and specification.
- 2.Production date, batch lot.
- 3.Product license, registration number, product technology requirement.
- 4.Sterilization expiry date.

8. CAUTION

- The product was sterilized and valid for three years, make sure the device is used in the period of validity.
- Do not use if elastomer fall off.
- Do not use if fixation band can't be fixed well.
- The device is designed and intended for SINGLE USE ONLY.
- Do not resterilize and/or reuse, otherwise patients could get the risk of infection. After use dispose of product and packaging in accordance with infectious medical waste disposal method.
- Sterilized by ethylene oxide. Do not use if the package damaged.




Caution



Use by date



Keep dry

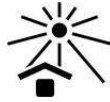

Keep away from sunlight

REF

Catalogue number



Date of manufacture



Manufacture



Do not use

STERILE EO

Sterilized Using EO



Do not use if package is damaged

LOT

Batch code



BrosMed Medical B.V.

ADDRESS: Mgr. Buckxstraat 8, 6134 AP Sittard, the Netherlands

TEL: +31 850140901

Mail: cs@brosmed.com



Brun Medical Co., Ltd.

ADDRESS: Building B8, Entrepreneurship and Innovation Park, 88 Changtang Road,
Yunlong Demonstration Zone, Zhuzhou 412000 HUNAN, CHINA

TEL: +0731-22029666

WEB: www.brunmd.com

Date of issue: 2022/5/5