

Manifold [INSTRUCTION FOR USE]

1. INTENDED PURPOSE

The product is intended to establish multiple channels and provide liquid medicine or contrast medium injection as a connecting channel by professional physician.

2. CONTRAINDICATION

No absolute contraindications when used correctly by medical personnel.

3. HANDING AND STORAGE

Stored in the relative humidity of not more than 80%, non-corrosive gases, a common, dry, well-ventilated and clean environment.

4. STRUCTURE AND SPECIFICATIONS

The device is consist of the luer connector, valve body, handle, protective cap.Specifications:

BR-MF1-OFF	BR-MF1-ON
BR-MF2-OFF	BR-MF2-ON
BR-MF3-OFF	BR-MF3-ON

5. PRODUCT PERFORMANCE

- Designed to withstand 500psi pressure.
- Multiple channels for multiple infusion therapy.
- Adopt the transparent materials, manifolds' flow path are clearly visible in use.

6. INSTRUCTION FOR USE

- 1.Remove the packaging, do not use if the package damaged.
- 2.Take out manifolds, and take off protective cap, then connect lines according to the regulation, check all the connections to make sure not liquid and air leakage.
- 3.According to patient needs, infuse appropriate medical liquid to implement diagnosis and treatment.
- 4.Single use only, discards after use.

7. PRODUCT LABEL AND PACKAGING (See the label or packaging)

- 1.Product name, model and specification.
- 2.Production date, batch lot.
- 3.Product license, registration number, product technology requirement.
- 4.Sterilization expiry date.

8. CAUTION

1. Procedure must be performed by trained medical personnel.
2. Sterilization is valid for three years, please confirm date of sterilization before use,do not use if expiry.
3. The device is designed and intended for SINGLE USE ONLY.
4. Do not resterilize and/or reuse, otherwise patients could get the risk of infection.
- 5.Sterilized by ethylene oxide. Do not use if the package damaged.




Caution



Use by date



Keep dry

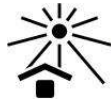

Keep away from sunlight

REF

Catalogue number



Date of manufacture



Manufacturer



Do not use

STERILE EO


Sterilized Using EO



Do not use if package is damaged

LOT

Batch code

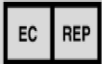

BrosMed Medical B.V.

ADDRESS: Mgr. Buckxstraat 8, 6134 AP Sittard, the Netherlands

TEL: +31 850140901

Mail: cs@brosmed.com



Brun Medical Co., Ltd.

ADDRESS: Building B8, Entrepreneurship and Innovation Park, 88 Changtang Road,
Yunlong Demonstration Zone, Zhuzhou 412000 HUNAN, CHINA

TEL: +0731-22029666

WEB: www.brunmd.com

Date of issue: 2022/5/5