

Balloon Inflation Device [INSTRUCTION FOR USE]

1. INTENDED PURPOSE

The device is intended to pressurize the balloon dilatation catheters in PTCA that create pressure in the balloon to dilate the blood vessel or indwelling stent in the blood vessel by professional physician.

2. CONTRAINDICATION

No absolute contraindications when used correctly by medical personnel.

3. HANDING AND STORAGE

Stored in the relative humidity of not more than 80%, non-corrosive gases, a common, dry, well-ventilated and clean environment.

4. STRUCTURE AND SPECIFICATIONS

The device is consist of the luer connector, connecting tubing, pressure gauge, syringe barrel, handle and three-way stopcock.

Specifications:

specification	Volume(ml)	Pressure(atm)
BR-BID-I30	20	30
BR-BID-I30L	30	30
BR-BID-II26	20	26
BR-BID-II30	20	30
BR-BID-III30	20	30
BR-BID-IV30	20	30

5. PRODUCT PERFORMANCE

The Balloon Inflation Devices can create and monitor pressure in the balloon, and can deflate the balloon quickly.

6. INSTRUCTION FOR USE

1. Remove the packaging, do not use if the package damaged.
2. Attach connecting tube to luer port of the prepared balloon catheter. Pressure can be increased by pushing the plunger handle, to check whether the device is leakage, or pressure gauge is working correctly, or balloon is broken.
3. Aspirate contrast agent into syringe, and eliminate any air that may have collected in the luer port. Create pressure in the balloon though injection of contrast agents.

7. PRODUCT LABEL AND PACKAGING (See the label or packaging)

- 1.Product name, model and specification.
- 2.Production date, batch lot.
- 3.Product license, registration number, product technology requirement.
- 4.Sterilization expiry date.

8. CAUTION

- Procedure must be performed by trained medical personnel.
- Sterilization is valid for three years, please confirm date of sterilization prior to use, do not use if expiry.
- Do not use if pressure gauge doesn't work when pressuring.
- Under working conditions the pressure difference should not exceed the full-scale value of the gauge.
- It's forbidden to use gas as a medium for creating the pressure in the balloon.
- The device is designed and intended for ONE-TIME USE ONLY.
- Do not resterilize and/or reuse, otherwise patients could get the risk of infection.
- After use dispose of product and packaging in accordance with infectious medical waste disposal method.




Caution



Use by date



Keep dry

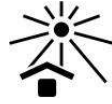

Keep away from sunlight

REF

Catalogue number



Date of manufacture



Manufacturer



Do not use

STERILE EO

Sterilized Using EO



Do not use if package is damaged

LOT

Batch code



BrosMed Medical B.V.

ADDRESS: Mgr. Buckxstraat 8, 6134 AP Sittard, the Netherlands

TEL: +31 850140901

Mail: cs@brosmed.com



Brun Medical Co., Ltd.

ADDRESS: Building B8, Entrepreneurship and Innovation Park, 88 Changtang Road,
Yunlong Demonstration Zone, Zhuzhou 412000 HUNAN, CHINA

TEL: +0731-22029666

WEB: www.brunmd.com

Date of issue: 2022/5/5